Intended Use

The Nasus Nasal Spray Device is intended for Over-The-Counter Use.

Indication for use

The Taffix™ (Powder) Spray Device is indicated for use as a protective mechanical barrier against allergens and viruses within the nasal cavity.

What is Taffix™?

Taffix™ is used to block viruses within the nasal cavity, viruses that could penetrate the respiratory system and cause infection.

Taffix was found highly effective in blocking several respiratory viruses including SARS-CoV-2 in laboratory studies.

To protect against airborne viruses, Taffix™ should be used prior to exposure to a high risk environment where viruses may be airborne, such as public place (e.g., supermarkets, pharmacies, school, sport club, flight, etc.) or encounters with anyone outside of the isolated members of the household.

Taffix™ may be applied up to 3 times over the course of the day.

How does Taffix™ work?

Taffix™ is a powder nasal spray, the powder turns to a gel in the nose which creates an active barrier that shields, protects and deactivates airborne viruses found often in circulated air in open spaces, planes, trains, large shopping centers and public transportation.

If concurrent use of another nasal spray is required, Taffix™ should be re-applied afterwards so that the barrier created is not disturbed.

Taffix™ can be used in addition to other treatments, as part of combination therapy strategy in the fight against viruses. For example, Taffix™ may be used in addition to donning a mask and wearing gloves.

Taffix™ is not a drug and does not cause drowsiness.

Who can use Taffix™?

- Taffix™ is suitable for adults, athletes, and children from 12 years old.

Contraindications:

- Do not use this product if you are sensitive to cellulose, citric acid or Benzalkonium chloride.
- Consult a physician before using Taffix™ in children below age 12 or pregnant or breast-feeding women

⚠️ Cautions:

- Discard Taffix™ bottle on the expiration date printed on the bottle or 6 month after first opening - whichever comes first.
- To avoid contamination do not use this product for more than one person.
- May be used up to 3 times a day.
- Keep out of reach of young children.
- Avoid contact with open wounds.
- Avoid contact with eyes. If powder gets into your eye, rinse with water.
- Do not rinse the bottle with water or other liquid as this may block the bottle.
- Do not use if tamper evident seal is broken.
- Always close the cap.
- Very few cases of allergic reactions to this product have been reported. Most reports received have been blocked nose, runny nose, sneezing, sore throat. If you experience reactions more severe than these - then use the of the product should be discontinued immediately and if symptoms persist, consult your physician.
- Remember to squeeze the bottle gently to avoid excess powder emittance.

How to store Taffix™?

- Store at room temperature or between 5°C ~ 30°C (41°F ~ 86°F).
- Do not refrigerate or freeze.

How to use Taffix™?

To defend against airborne viruses, Taffix™ may be applied up to 3 times per day to maintain an effective barrier against viruses.

Remember - Taffix™ bottle cannot be shared with other people. Mark your bottle and avoid sharing it with other people to prevent potential transmission of viruses or germs.

Taffix™ bottle contains 1000 mg of powder (about 200 puffs). Once applied, Taffix™ becomes effective after a minute and is effective for approximately 5 hours.

Before you start using Taffix™ - you can practice by squeezing the bottle gently aiming it into the air and then follow the using instructions:


How to prepare a new Taffix™ for use?

- Take your Taffix™ Nasal Spray out of the package and open the cap
- Shake Taffix™ bottle
- Test spray into the air twice
- The Taffix™ is now ready for use

How to use the Taffix™?

Every time you take Taffix™ follow the instructions below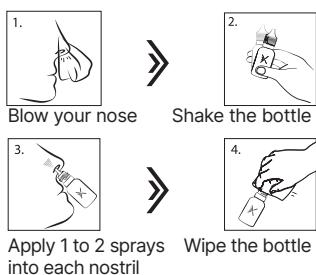

During first use, you may experience a tingling or burning sensation in the nose especially if the nasal tissue is sensitive or inflamed.

You may also feel some form of irritation, especially if your nasal membranes are already inflamed or irritated. This will subside quickly.

Some individuals can feel a slight sensation or blocked nose, this is normal and is the feel of the barrier created by Taffix™. If you experience continuous discomfort you can wash your nose with warm water until relief. If you experience serious discomfort after application of Taffix™ consult your physician.

Taffix™ bottle should be used for one person only, sharing a bottle with other people could expose them to infection!!

Chemical Composition: Hypromellose (HPMC) (89.9%), Citric Acid (6%), Sodium Citrate (4%), Benzalkonium Chloride (0.1%) and Menthol (<0.1%).

Taffix™ can be bought in several configurations:

- [REF]** 1841600 Taffix™ - 1 bottle of 1g powder
- [REF]** 1841602 Taffix™ Travel - 1 bottle of 0.7g powder
- [REF]** 1841603 Taffix™ Family - 4 bottles of Taffix™ (**[REF]** 1841600)

Nasus Pharma Ltd.

Harakevet 29 Tel Aviv, 6618306, Israel

Tel: +9723-573-6632

Fax: +9723-573-6664

Email: info@nasuspharma.com

Website: www.nasuspharma.com

Made in Israel

Registration number In Israel MOH (AMAR): 33010001

Mdi Europa GmbH

Langenhagener Str. 71, D-30855

Langenhagen, Germany

Tel: +49 511 39089530

Email: info@mdi-europa.com

Website: mdi-europa.com

Symbols Glossary:

	Legal Manufacturer
	CE Mark, indicating that the device complies with Council Directive 93/42/EC (MDD)
	Caution, consult accompanying documents
	Consult instructions for use
	Batch Number
	Catalogue number
	Authorized representative in the European Community
	Do not use if package is damaged
	Minimum and maximum storage temperatures
	Keep away from sunlight
	Expiry date (In format: YYYY-MM-DD)
	Keep dry

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EC REP Mdi Europa GmbH